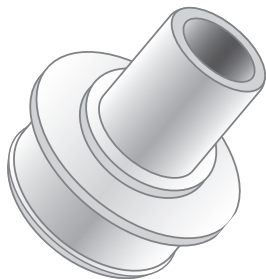
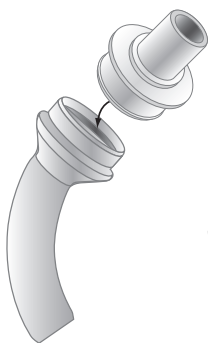
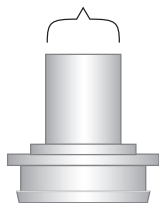
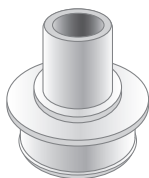
PROVOX

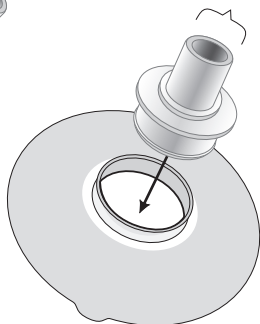
BasePlate Adaptor

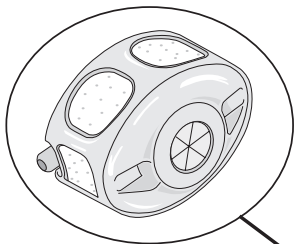
ISO 15 mm




ISO 15 mm



TrachPhone®

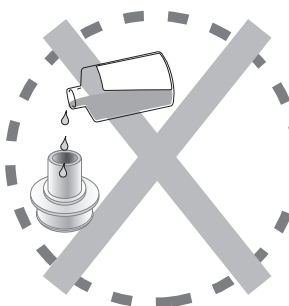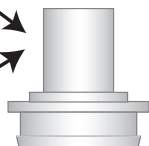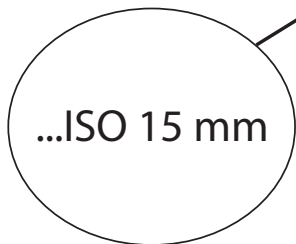
| | |
|--------|--------|
| Oil | Olje |
| Öl | Olej |
| Huile | Ulei |
| Olio | Ulje |
| Olje | Ulje |
| Olie | Έλαιο |
| Aceite | Yağ |
| Olja | Ξηλη |
| Nafta | Масло |
| Óleo | Minyak |
| Olía | Minyak |
| Olía | オイル |
| Öli | 오일 |
| Ejja | 油 |
| Alyva | น้ำมัน |
| Olej | Dầu |
| Olaj | שמן |
| Olej | الزيت |
| | روغن |

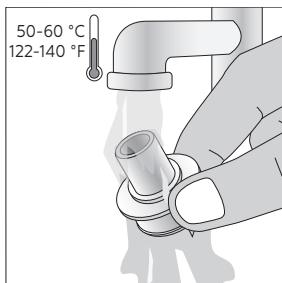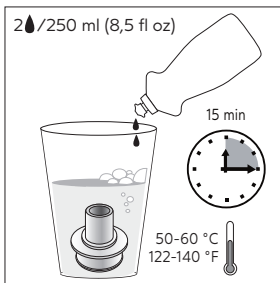
1.

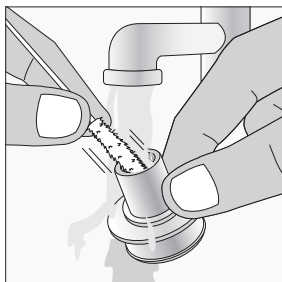
2.

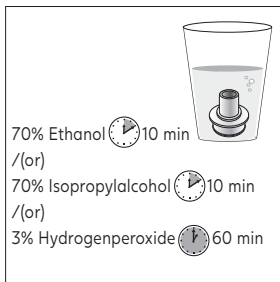3.

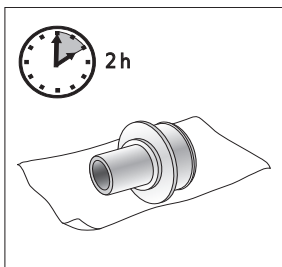4.

5.

4

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Provox BasePlate Adaptor

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles ; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ûhilduvad tooted; Saderigie izstrādājumi; Suderinami gaminiai; Kompatibilni produkti; Kompatibilis termékek ; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Prognose compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Uyumlu Ürünler; Համատեղելի սարքեր; Совместимые изделия; Produk yang Kompatibel; Produk yang Sesuai; 互換性のある製品; 호환 가능한 제품; 相容產品; ผลิตภัณฑ์ที่เข้ากันได้; Sản phẩm tương thích ; מוצרים תואמים; المنتجات المتوافقة; محصولات سازگار

| | Provox BasePlate Adaptor |
|------------------------------|--------------------------|
| Provox LaryTube | X |
| Provox LaryButton | X |
| Provox StabililBase | X |
| Provox XtraBase | X |
| Provox StabililBase OptiDerm | X |
| Provox Flexiderm | X |
| Provox Optiderm | X |
| TrachPhone | X |
| Freevent DualCare | X |
| Freevent XtraCare | X |

Contents

| | |
|-----------------------------|----|
| EN - ENGLISH | 7 |
| DE - DEUTSCH | 9 |
| NL - NEDERLANDS..... | 11 |
| FR - FRANÇAIS..... | 13 |
| IT - ITALIANO..... | 15 |
| ES - ESPAÑOL..... | 17 |
| PT - PORTUGUÊS..... | 19 |
| SV - SVENSKA..... | 21 |
| DA - DANSK..... | 23 |
| NO - NORSK..... | 25 |
| FI - SUOMI..... | 27 |
| IS - ÍSLENSKA | 29 |
| ET - EESTI..... | 31 |
| LV - LATVIEŠU | 33 |
| LT - LIETUVIŲ KALBA..... | 35 |
| CS - ČESKY..... | 37 |
| HU - MAGYAR..... | 39 |
| SK - SLOVENČINA..... | 41 |
| SL - SLOVENŠČINA..... | 43 |
| PL - POLSKI | 45 |
| RO - ROMÂNĂ..... | 47 |
| HR - HRVATSKI | 49 |
| SR - SRPSKI..... | 51 |
| EL - ΕΛΛΗΝΙΚΑ | 53 |
| TR - TÜRKÇE..... | 55 |
| HY - ՀԱՅԵՐԷՆԵՆ..... | 57 |
| RU - РУССКИЙ..... | 59 |
| ID - BAHASA INDONESIA | 61 |
| MS - BAHASA MELAYU..... | 63 |
| JA - 日本語..... | 65 |
| KO - 한국어..... | 67 |
| ZHTW - 繁體中文 | 69 |
| TH - ไทย..... | 71 |
| VI - TIẾNG VIỆT..... | 73 |
| HE - עברית..... | 76 |
| AR - العربية..... | 78 |
| FA - فارسی..... | 80 |

Intended Use

Provox BasePlate Adaptor is an accessory that allows attaching medical devices, e.g. a HME, with an ISO 15 mm standard connector to a Provox attachment.

Contraindications

Shall not be used for mechanical ventilation.

Description of the device

Provox BasePlate Adaptor ("adaptor") is an accessory product for rehabilitation after total laryngectomy. It allows attaching medical devices, (HME), with ISO 15mm standard connector to a tracheostoma by fitting it into a Provox Adhesive, Provox LaryButton or Provox LaryTube. A typical example would be to attach an HME with built-in oxygen adapter (TrachPhone). For single patient use.

WARNINGS

- Unconscious patients must be constantly monitored. Attached devices may inadvertently slide off the adaptor, or the base plate may detach from the tracheostoma.
- Make sure the adaptor is free from oil or any other lubricant before and during use. Lubrication may lead to detachment of devices from the adaptor, or of the adaptor from the base plate.

PRECAUTIONS

- Read and follow the instructions of the medical device to be used with the adaptor.
- Never use force while attaching or detaching devices or the adaptor itself.
- Do NOT sterilize! The device is for single patient use only.
- Do not use if the device shows any signs of cracking or structural damage.

Cleaning and maintenance

Clean and disinfect the device before each use. Use drinking water to clean and rinse the device.

- Rinse the device in water (Fig. 1).
- Place the device in water-based mild soap solution for 15 minutes (Fig. 2).
- Clean inside of the device with Provox TubeBrush or Provox Swab. Rinse the device with water (Fig. 3).

- Disinfect the device at least once a day with one of the following methods: Ethanol 70% for 10 minutes or Isopropylalcohol 70% for 10 minutes or Hydrogenperoxide 3% for 60 minutes (Fig. 4).
- Let the device dry at least for 2 hours or until it looks dry (Fig. 5).

CAUTION: Do not use the device until it is completely dry. Inhalation of disinfectant fumes can cause severe coughing and irritation of the airways. Provox BasePlate Adaptor shall be cleaned and disinfected before each use to avoid risk of contamination. The device is for single patient use only, reuse between patients may cause cross contamination.

During hospitalization it is important to clean and disinfect Provox BasePlate Adaptor immediately after use and again just before use due to an increased risk of infection and device contamination. Use sterile water for rinsing, rather than drinking water.

Device lifetime

Provox BasePlate Adaptor may be used for a maximum period of 3 months. If the device shows any signs of damage it shall be replaced earlier.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2024 Coloplast A/S. All rights reserved.
Atos Medical AB