
TrachPhone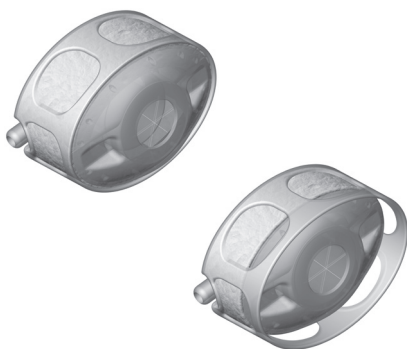

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Contents

EN - ENGLISH.....	4
DE - DEUTSCH.....	5
NL - NEDERLANDS.....	7
FR - FRANÇAIS.....	8
IT - ITALIANO.....	10
ES - ESPAÑOL.....	11
PT - PORTUGUÊS.....	13
SV - SVENSKA.....	14
DA - DANSK.....	16
NO - NORSK.....	17
FI - SUOMI.....	19
IS - ÍSLENSKA.....	20
ET - EESTI.....	21
LT - LIETUVIŲ KALBA.....	23
CS - ČESKY.....	24
HU - MAGYAR.....	26
SK - SLOVENČINA.....	27
SL - SLOVENŠČINA.....	29
PL - POLSKI.....	30
RO - ROMÂNĂ.....	32
HR - HRVATSKI.....	33
SR - SRPSKI.....	35
EL - ΕΛΛΗΝΙΚΑ.....	36
TR - TÜRKÇE.....	38
HY - ՀԱՅԵՐԵՆ.....	39
RU - РУССКИЙ.....	41
ID - BAHASA INDONESIA.....	42
MS - BAHASA MELAYU.....	44
JA - 日本語.....	45
KO - 한국어.....	47
ZHTW - 繁體中文.....	49
TH - ไทย.....	50
VI - TIẾNG VIỆT.....	52
HE - עברית.....	55
AR - العربية.....	57
FA - فارسی.....	59

Intended use

For patients breathing spontaneously via an ET tube or a tracheostomy tube in the hospital or at home.

Description of the device

TrachPhone heats and humidifies the inhaled air and partially restores breathing resistance.

It can be occluded with a finger to facilitate speech. After release the valve will open automatically. TrachPhone is connected to an ISO 15 tube. An integrated suction port makes it possible to clean the tracheostomy tube from mucus as needed. An oxygen tubing can be connected via the oxygen connector present on TrachPhone.

Technical data

Tidal volume range	>50 ml
Dead space	9.5 ml
Pressure drop* at - 30 l/min - 60 l/min	20 Pa 50 Pa
Moisture loss* at - VT= 500 ml - VT= 1000 ml in HME mode	17 mg/l 18 mg/l
Weight	2.9 g
Length	22 mm
Height	28 mm
Width	38 mm
Non-conductive	

* According to ISO 9360

Connections

15 mm female connector for ET tube or tracheostomy tube. 4 mm connector for supplemental oxygen supply. Use appropriate size of oxygen tubing. The TrachPhone has a valve for forced inspiration/expiration or suctioning.

Instructions for use

The device should be placed at the open end of the ET tube or tracheostomy tube. Check that the connection is properly fixed. If supplemental oxygen is required, connect an oxygen tube to the oxygen port. Normal efficiency is reached after a few breaths. Should be replaced every 24 hours or as required, to avoid resistance increase due to accumulation of secretions.

Before use, check product for deformity and valve function by closing the valve and releasing it. Do not use if the device shows any signs of defect. Do not use if the package is opened or damaged.

Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

Speech valve

The TrachPhone contains a valve with a spring that can easily be occluded with a finger facilitating speech. After releasing the finger, the valve will open automatically.

Contraindications

Do not use beyond recommended tidal volume range, as the added dead space may cause CO₂ retention at too low tidal volumes. A too high tidal volume may lead to unsatisfactory humidification.

Do not use on dehydrated patients or patients with very heavy secretions from the lungs and airways.

Caution

Single use only: do not sterilize or reuse, as the intended function will be affected, and this might increase the risk of possible transmission of infection. Destroy after use.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2025 Coloplast A/S. All rights reserved.
Atos Medical AB