
# FREEVENT<sup>®</sup>

## TrachPhone<sup>®</sup>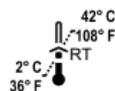

## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

Freevent® and TrachPhone® are registered trademarks owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page [www.atosmedical.com/patents](http://www.atosmedical.com/patents).

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Fabricante; Tillverkare; Producent; Producers; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель; Producers; Pengilang; 製造業者; 제조사; 製造商; 制造商; תאריך ייצור; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje; Data produkcji; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; წარმოების თარიღი; Ἀρτοποιημάτων ἀντιστοιχία; İstehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 製造日期; תאריך ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käyttöpäivä; Sīdasti notkunardagur; Kōlblik kuni; Derīguma termiņš; Naudoti iki; Datum expirace; Felhasználhatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на годност; Son kullanim tarihi; ვარგობის; Ἡχρηστέληρησῶν ἡμερῶν; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; תאריך אחרון לשימוש; تاريخ انتهاء الصلاحية

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## Indications

For patients breathing spontaneously via an ET tube or a tracheostomy tube in the hospital or at home.

## Technical data

Recommended tidal volume range	50 - 1000 ml
Compressible volume	9.5 ml (dead space)
Moisture output*	V <sub>T</sub> = 250 ml: 23.4 mg/l V <sub>T</sub> = 500 ml: 20.5 mg/l V <sub>T</sub> = 1000 ml: 16.6 mg/l
Resistance to flow	35 ± 15 Pa at 30 l/min
Weight	2.9 g
Length	22 mm
Height	28 mm
Width	38 mm
Non-conductive	

\*MDA Report 01052

## Connections

15 mm female connector for ET tube or tracheostomy tube. 4 mm connector for supplemental oxygen supply. Use appropriate size of oxygen tubing. The TrachPhone has a valve for forced inspiration/expiration or suctioning.

## Instructions for use

The device should be placed at the open end of the ET tube or tracheostomy tube. Check that the connection is properly fixed. If supplemental oxygen is required, connect an oxygen tube to the oxygen port. Normal efficiency is reached after a few breaths. Should be replaced every 24 hours or as required, to avoid resistance increase due to accumulation of secretions.

Before use, check product for deformity and valve function by closing the valve and releasing it. Do not use if the device shows any signs of defect. Do not use if the package is opened or damaged.

## Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

## Speech valve

The TrachPhone contains a valve with a spring that can easily be occluded with a finger facilitating speech. After releasing the finger, the valve will open automatically.

## Contraindications

Do not use beyond recommended tidal volume range, as the added dead space may cause CO<sub>2</sub> retention at too low tidal volumes. A too high tidal volume may lead to unsatisfactory humidification.

Do not use on dehydrated patients or patients with very heavy secretions from the lungs and airways.

## Caution

Single use only; do not sterilize or reuse, as the intended function will be affected, and this might increase the risk of possible transmission of infection. Destroy after use.

## Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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